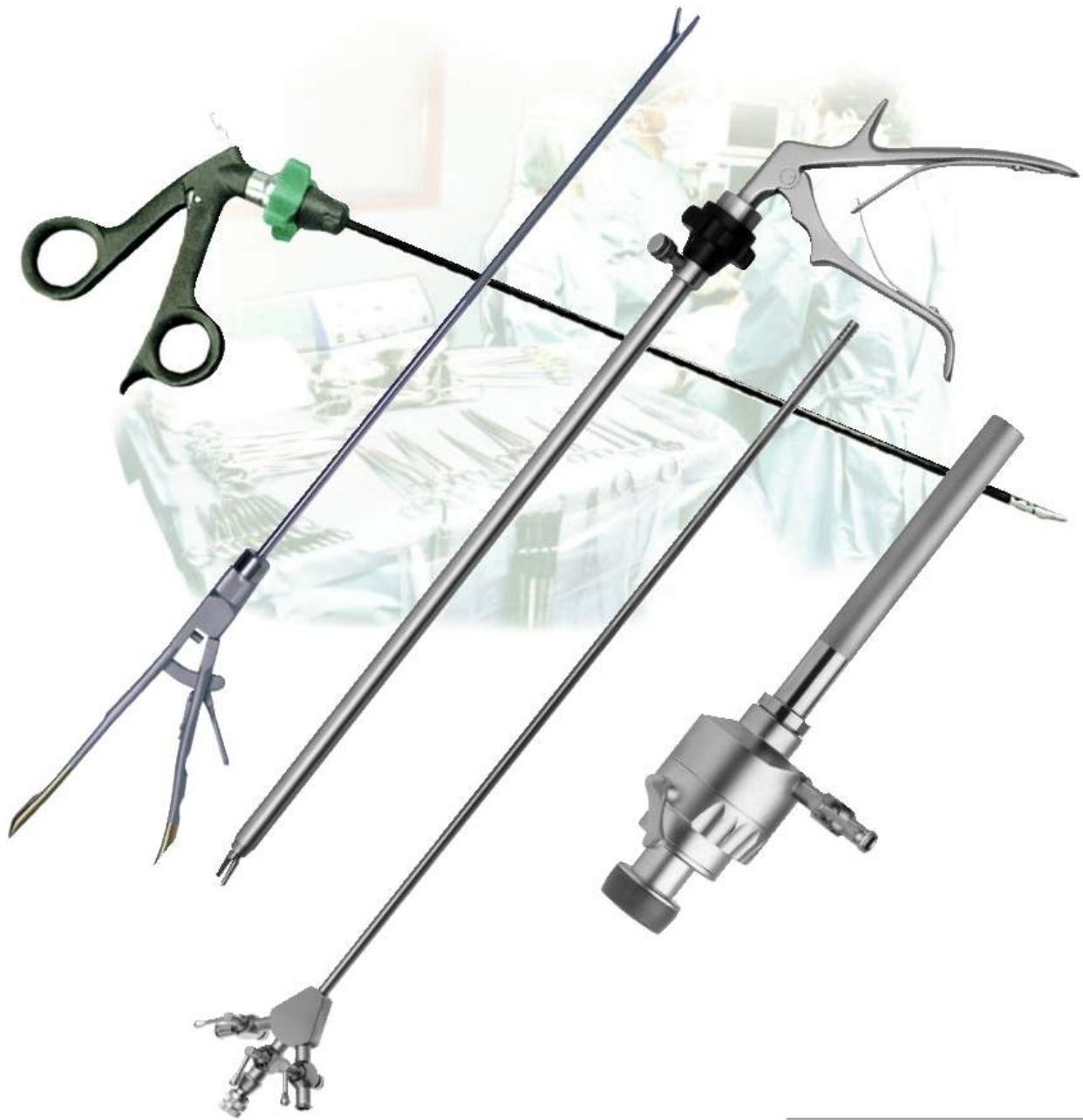
LAPAROSCOPY

LAPAROSCOPY

Instrumenteneinsatz

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Instrument-insert

Wide range of instruments and cost-savings exchange in case of use up by exchangeable inserts for Ø 3 mm, Ø 5 mm und Ø 10 mm shafts.

Distal-Interface

In this place, the insert can be screwed into the shaft.

Shafts

For this hand instrumentssystem, isolated and metall shafts are available in diameter of Ø 5 mm and Ø 10 mm. Another advantage is that we have also shafts with Ø 10 mm outer diameter which is made for the Ø 5 mm inserts. Therefore, you will receive a very robust instrument..

Proximal-Interface

This interface is same by all handles and shafts; because of this, all inserts are able to be connected with the whole handle-spectrum through shaft knob.

Handles

To our system, two different handle polymers made out of plastic are available - one with and one without ratchet.

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The instrument-inserts are pushed right down into the shafts and screwed to the end.

Important: close the insert-mouth completely, because of impairment cutting!

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The insert should be closed now and the handle should be opened at the maximum.

From now on, put the ball into the gap from a oblique position. Close the handle and screw the shaft knob.

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Now, the instrument is completely assembled. Concluding, a function control is to be held.

It will be tested if a relaxed connection between handle and insert exist.

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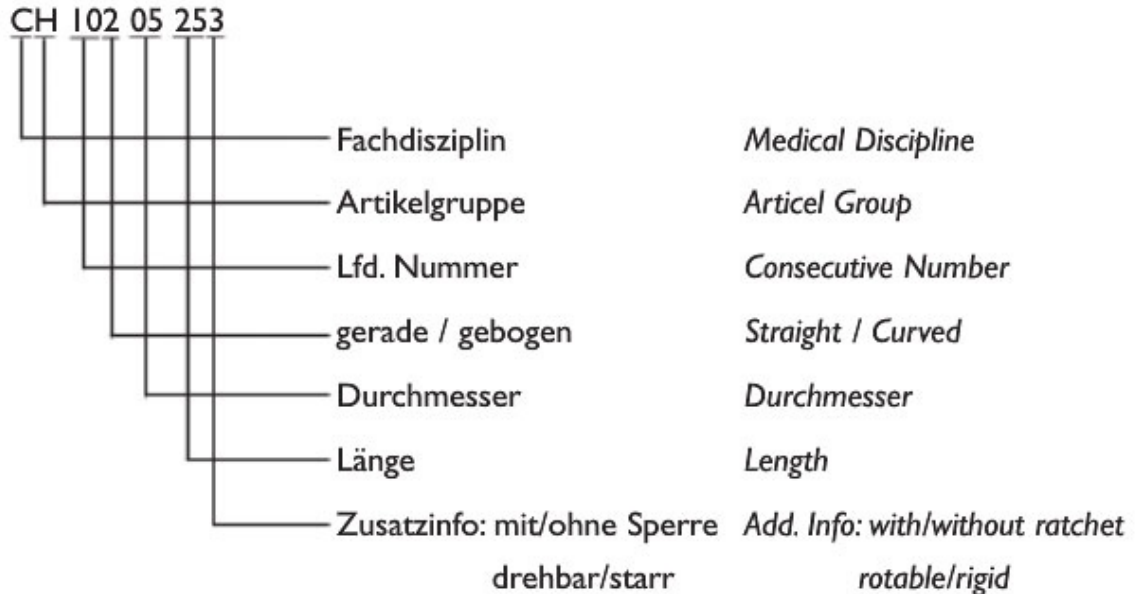
Handinstruments
for minimal invasive surgery

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